

# NON-DELIVERY PORTABLE OXYGEN SYSTEMS



## FREESTYLE

**ONLY  
4.4 lbs**



## INOGEN ONE

**ONLY  
9.7 lbs**



## EVERGO

**ONLY  
9.9 lbs**



## ECLIPSE

**ONLY  
17.4 lbs**



### USAGE INFORMATION

- Dispensed w/stationary system E1390
- Portable E system provided and billed
- FAA approved
- Must meet portability criteria

### USAGE INFORMATION

- Stationary E1390
- Portable E1392
- FAA approved
- Must meet portability criteria

### USAGE INFORMATION

- Stationary E1390
- Portable E1392
- FAA approved
- Must meet portability criteria

### USAGE INFORMATION

- Stationary E1390
- Portable E1392
- FAA approved
- Must meet portability criteria
- Bleed in for CPAP therapy

### \*SETTINGS INFORMATION

**Conservor Settings 1-3**  
1 = 3 1/2 hrs 2 = 2 1/2 hrs 3 = 2 hrs

**Battery Pack (Airbelt)**  
1 = 10 hrs 2 = 6 hrs 3 = 5 hrs

### \*SETTINGS INFORMATION

**Conservor Settings 1-5**  
1 = 3 hrs 20 mins 2 = 3 hrs  
3 = 2 hrs 40 mins 4 = 2 hrs 20 mins  
5 = 2 hrs

### \*SETTINGS INFORMATION

**Conservor Settings 1-6 Using Dual Batteries**  
1 = 12 hrs 2 = 8 hrs 3 = 6 hrs  
4 = 4 hrs 5 = 4 hrs 6 = 4 hrs

### \*SETTINGS INFORMATION

**Conservor Settings 1-6**  
1 = 4.4 hrs 2 = 3.6 hrs 3 = 3 hrs  
4 = 2.6 hrs 5 = 2.3 hrs 6 = 2.1 hrs

**Continuous Settings 1-3**  
1 LPM = 3.7 hrs 2 LPM = 2.4 hrs  
3 LPM = 1.3 hrs

\* If continuous order is 1 LPM = setting 1-3 2 LPM = setting 2-4 3 LPM = setting 3-5. Setting range allows user to adjust for activity/altitude related symptoms.

### POWER INFORMATION

- AC & DC (car adapter) Operation
- Internal Battery
- Battery Pack (Airbelt)

### POWER INFORMATION

- AC & DC (car adapter) Operation
- Internal Battery

### POWER INFORMATION

- AC & DC (car adapter) Operation
- Internal Battery

### POWER INFORMATION

- AC & DC (car adapter) Operation
- Internal Battery

# CARE MEDICAL & REHABILITATION EQUIPMENT - YOUR TRAVEL OXYGEN EXPERTS -

## SEATTLE, WASHINGTON

4135 Stone Way North - 98103  
**(206) 547-2200**

## EVERETT, WASHINGTON

3010 Grand Avenue - 98201  
**(425) 252-6456**

## TACOMA, WASHINGTON

1902 Tacoma Avenue South - 98402  
**(253) 474-1774**

## OLYMPIA, WASHINGTON

3430 Pacific Avenue SE #A5 - 98501  
**(360) 459-0999**

## LONGVIEW, WASHINGTON

925 Vandercook Way - 98632  
**(360) 636-3120**

## VANCOUVER, WASHINGTON

1720 NE Andresen Road - 98661  
**(360) 693-4777**

## PORTLAND, OREGON

1877 NE 7th Avenue - 97212  
**(503) 288-8174**

## SALEM, OREGON

1944 McGilchrist SE - 97302  
**(503) 378-1756**

## EUGENE, OREGON

1280 Charnelton - 97401  
**(541) 343-0304**



## CARE MEDICAL & REHABILITATION EQUIPMENT IS YOUR "PARTNER IN CARE"

Care Medical & Rehabilitation Equipment is a family-owned and operated company that has been providing quality home medical & respiratory equipment and services in Oregon and Washington since 1970.

[www.caremedical.com](http://www.caremedical.com)